

# BONE EASY QUICKSMILE

## FROM ZERO TO SMILE - IN JUST ONE APPOINTMENT



BONE EASY®

PREVIOUSLY, THE PROCESS OF DEFINING AND VALIDATING THE FINAL SMILE OF AN EDENTULOUS PATIENT REQUIRED MULTIPLE APPOINTMENTS AND LENGTHY LAB WORK. **WHAT IF YOU COULD ACHIEVE ALL REFERENCES FOR YOUR IMPLANT PROCEDURE IN JUST ONE SESSION?**



## MEET QUICKSMILE

AN INNOVATIVE KIT THAT SIMPLIFIES AND ACCELERATES THE SMILE PLANNING PROCESS



[info@boneeasy.com](mailto:info@boneeasy.com)


+351 929 013 061



## WORKFLOW STEPS

### 1 ESTHETIC SELECTION

PATIENT SELECTS THE SHAPE THEY PREFER

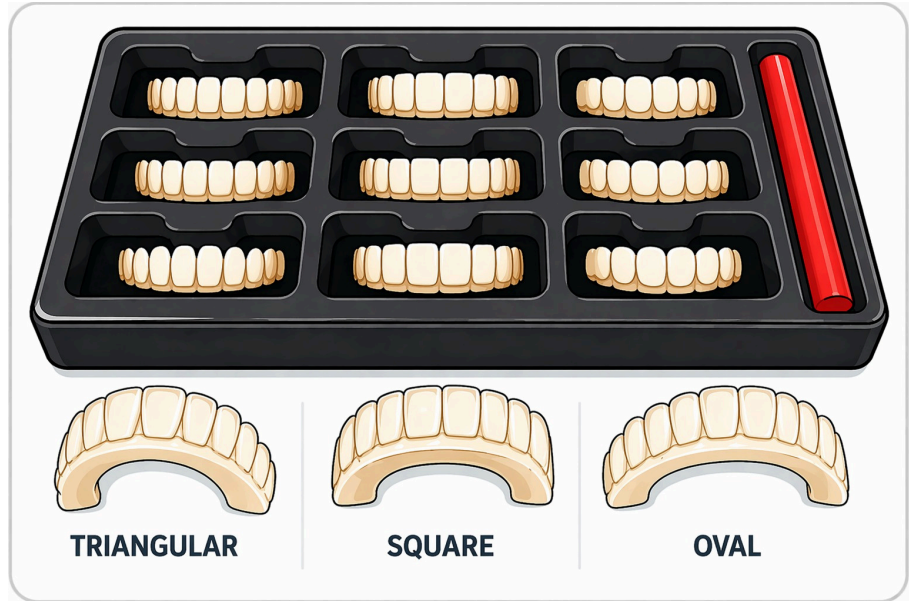

### 2 DIMENSION SELECTION

DENTISTS SELECT THE BEST ARCH SIZE



## WORKFLOW STEPS

### 3 TRAY SELECTION

PICK THE CORRESPONDING  
SCHRNEIDERMAKER TRAY



### 4 WAX ADDITION

APPLY **RED** WAX TO DEFINE THE HEIGHT



### 5 TEETH POSITIONING

PUSH THE CHOSEN TEETH INTO THE WAX



**WORKFLOW STEPS**

**6 SILICONE**

**APPLY SILICONE AND SEAT  
IN MOUTH**



**7 ADJUSTMENTS**

**ADJUST THE MEDIAL LINE  
AND THE OCCLUSION  
PLANE**



## WORKFLOW STEPS

### 8 PATIENT APPROVAL

TAKE A PICTURE TO SHOW THE PATIENT THE CHOSEN SMILE



### 9 CBCT SCAN

PATIENT UNDERGOES A CBCT WITH THE DEVICE ON



### 10 DIGITAL PLANNING



## WHY IS THIS THE BETTER ALTERNATIVE?

- NO WAITING. ALL-IN-ONE APPOINTMENT
- THE PATIENT SELECTS & APPROVES EVERYTHING
- RECEIVE ALL DIGITAL REFERENCES INSTANTLY



## TRADITIONAL TO QUICKSMILE

- |                         |   |                       |
|-------------------------|---|-----------------------|
| • DURATION 1 WEEK       | → | 1 APPOINTMENT         |
| • MULTIPLE APPOINTMENTS | → | SINGLE VISIT          |
| • LAB DEPENDENCE        | → | IN CLINIC- REFERENCES |
| • ADJUSTMENTS OVER TIME | → | REAL TIME ADJUSTMENTS |