User Manual- Manual de Utilizador
By RESDEV MED, LDA

Tailored implant

design your own implants

Introdução

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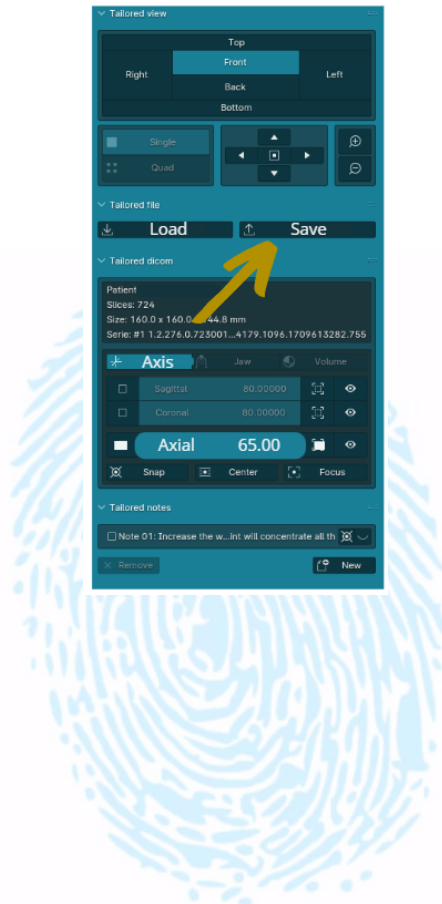
Introduction

Tailoredimplant is an interface between oral, maxillofacial and orthopedic surgery doctors and manufacturers of custom-made implants and grafts. At Tailoredimplant, surgeons have access to all decision-making procedures.

Our objective is to improve communication between surgeons and implant manufacturers, allowing them to view three-dimensional implants created and their relationships and interactions with the patient's anatomy.

Minimum system and installation requirements

Operating system: Windows 10.

Processor: Intel Core i7 or compatible.

RAM memory: 16GB.

Graphics card: Nvidia or ATI with minimum 2GB available.

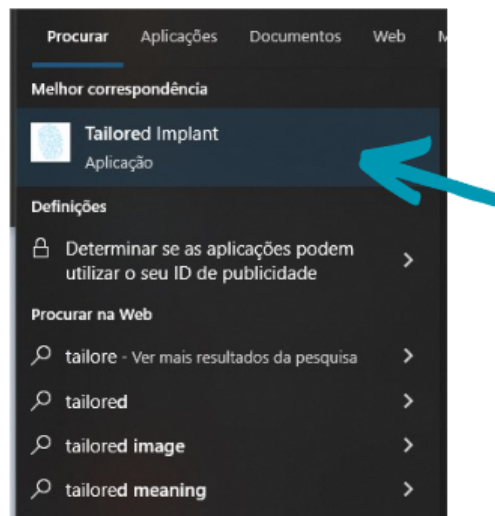
Monitor: 19 inches 1080px x 1920px.

Hard disk: 2GB free space.

Keyboard: normal with numpad.

Mouse: 3 buttons with scroll wheel

The download is done in the timeline of each product, at the top, or in the validation phase. After downloading the software, it is necessary to unzip and install it, once installed it appears listed in the programs.

Funcionalities

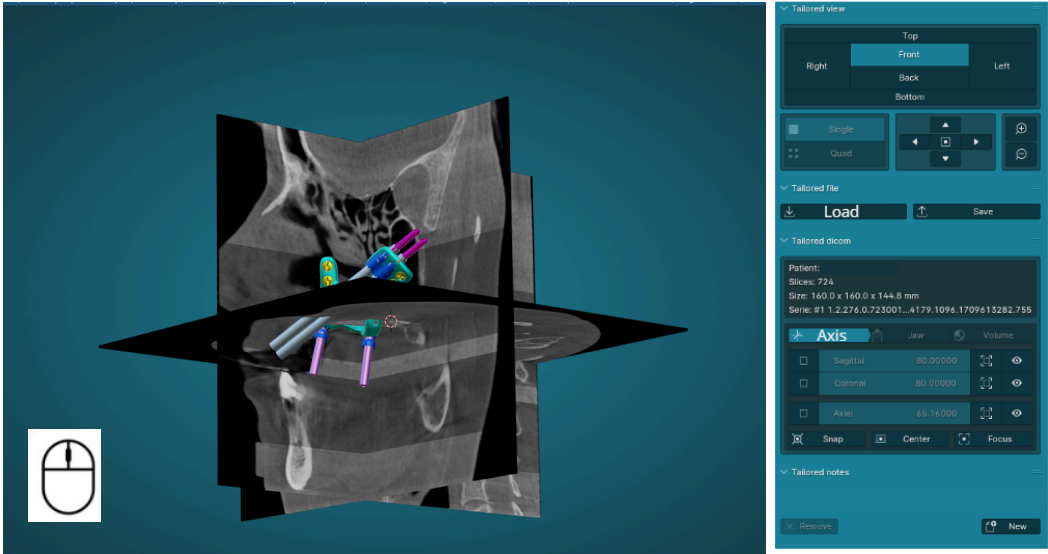
The software exclusively opens files with the .timp extension, generated by BoneEasy.

1. Files can be opened by double-clicking on the .timp icon. or using the upload option in the side tab, selecting the desired case.


How to use

The view using the software can be performed through the sagittal, coronal and axial planes by pressing the mouse wheel (1)

(1)

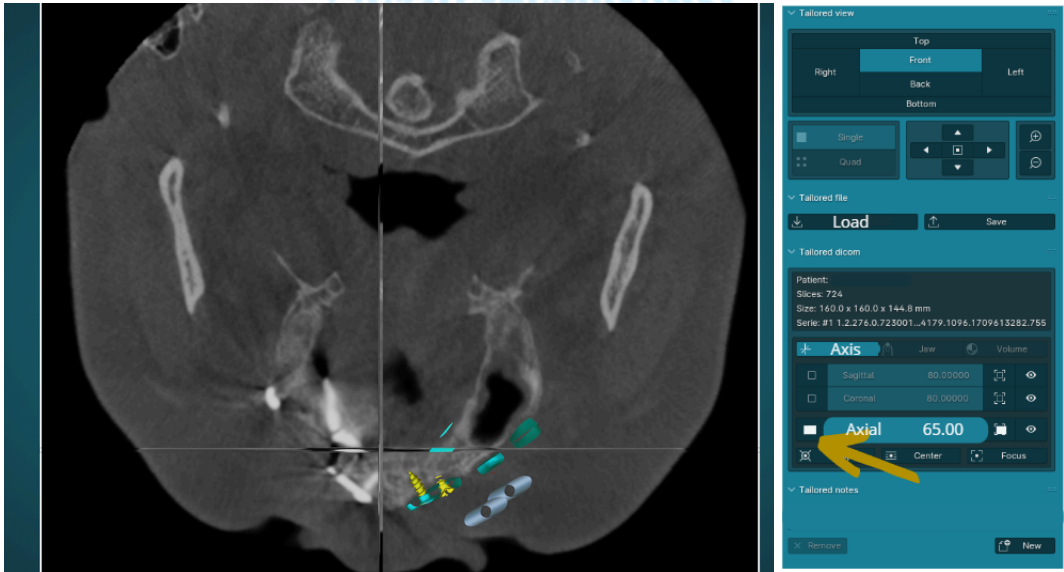or we can have a sectional view of the implant superimposed on the planes (2)

(2)

Add notes

In Tailoredimplant it is possible to add notes on modifications that are requested by the surgeon.

To do this, select the right mouse button on the point requesting change.

After identifying the location, select New, present in the side tab and fill in the modifications to be applied.

Click OK and a list will appear in the side tab.

Recording and sending

After viewing the project and making notifications for modifications, file must be saved using the command on the Save side tab.

This file is then attached to Dentlabsoft platform at the timeline of the case evaluated. If any type of change is not necessary, only carry with the validation phase on the Dentlabsoft platform.